


**EUROPLEX®**
O

A high quality orthopedic material: moldable, resilient, convenient, hygienic.



EUROPLEX® O is a proven material for the manufacturing of orthopedic devices. The advantages can be summed up as follows:

- EUROPLEX® O is characterized by its excellent impact and notched impact strength as well as its rigidity. Insoles and other orthopaedic devices made from EUROPLEX® O are virtually unbreakable.
- EUROPLEX® O can be thermoformed from temperatures as low as 90 °C without predrying. It is easy to machine and its shape can also be subsequently corrected by simple means.
- EUROPLEX® O is not attacked by perspiration and is generally highly resistant to chemicals.
- The relatively light inherent color of EUROPLEX® O makes orthopedic aids very unobtrusive to wear and enables pressure points to be easily identified during fitting.
- EUROPLEX® O shows very good X-ray transmission. That means there is no need to remove orthopaedic devices for this examination method.
- EUROPLEX® O is physiologically unobjectionable and has a perfectly smooth surface. It is therefore easy-care and hygienic.

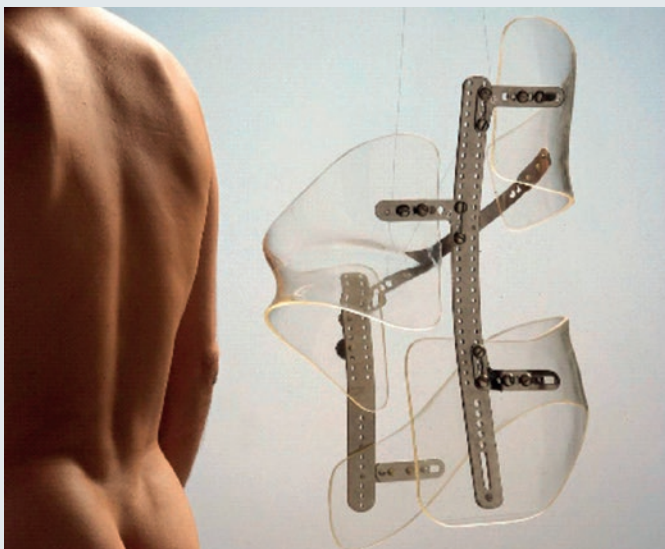

Transparent EUROPLEX® O sheets are available in size 2000x1250 mm and thicknesses between 1.5 bis 4.0 mm ex stock for short-term delivery.

Transparent insole and orthosis manufactured from EUROPLEX® O sheets.

Product Properties	EUROPLEX® O	Unit	Standard
Density	1.15	g/cm ³	ISO 1183
Glass transition temperature	92	°C	ISO 11357
Max. permanent service temperature	70	°C	
Modulus of elasticity	2,750	MPa	ISO 527
Tensile strength	80	MPa	ISO 527

Processing instructions for EUROPLEX® O sheets:

Sawing: Sawing should be performed using unset, carbide-tipped circular sawblades with the largest possible number of teeth (72–90). Take care to feed the material at a uniform rate, but not too slowly. With band saws, use slightly side-set blades with 14 teeth per inch (as for non-ferrous metals).

Milling: Clean cuts are achieved with single-edged endmilling cutters (plunge milling). The speed of the spindle should be about 11000 rpm.

Drilling: For drilling you may use the commercially available twist drills designed for metal.

Die cutting: EUROPLEX® O can be die-cut without difficulty and does not need to be pre-heated.

Grinding: The best results are achieved with belt sanders of grit 240 at a belt speed of 8 to 10 m/s. Do not press the workpiece against the belt for too long or with too much force, but lift it briefly at intervals.

Polishing: Polishing is best performed with a buffing wheel made of grey cotton or flannel. Before starting to polish, apply some wax to the rotating wheel. The peripheral speed should be around 20 m/s. Once again: do not press the material to the polishing wheel for too long or with too much force, but lift it briefly at intervals.

Barrel polishing: Generally speaking, barrel polishing is also possible. Depending on the original surface quality, the material has to be kept in the barrel polishing unit for up to 24 hours.

Thermoforming: EUROPLEX® O has a glass transition temperature of 92 °C. Optimum thermoforming results are achieved at between 100 and 120 °C. If the temperature is further increased, this results in irreversible clouding of the material, which does not affect its mechanical properties, however. Pre-drying is not required. Since the thermoforming conditions vary widely from fabricator to fabricator, the degree of clouding may be regarded as a certain indicator of the optimum conditions (e.g. time span to onset of clouding under IR-radiator – 0.5 min.). The following data are meant as a guideline and will have to be adjusted to the actual conditions.

Heating by IR-radiator: At a distance of about 20 to 30 cm from the radiator, a heating period of about 2 min. is sufficient.

Heating in an air-circulation oven: At a preselected temperature of about 150 °C, the heating periods are approximately the following:

- 2.0 mm thick 2.0 – 2.5 min.
- 3.0 mm thick 3.0 – 3.5 min.
- 4.0 mm thick 4.0 – 4.5 min.

Bonding: Best suited for bonding are products such as epoxy, cyanoacrylate or PUR-type adhesives.

POLYVANTIS GmbH Riedbahnstraße 70 **www.plexiglas.de**
64331 Weiterstadt **www.polyvantis.com**
Germany

® = registered trademark

EUROPLEX is a registered trademark of POLYVANTIS GmbH, Germany.
Certified to DIN EN ISO 9001 (Quality) and DIN EN ISO 14001 (Environment)

This information and all further technical advice is based on our present knowledge and experience. However, it implies no liability or other legal responsibility on our part, including with regard to existing third party intellectual property rights, especially patent rights. In particular, no warranty, whether express or implied, or guarantee of product properties in the legal sense is intended or implied. We reserve the right to make any changes according to technological progress or further developments.

The customer is not released from the obligation to conduct careful inspection and testing of incoming goods. Performance of the product described herein should be verified by testing, which should be carried out only by qualified experts in the sole responsibility of a customer. Reference to trade names used by other companies is neither a recommendation, nor does it imply that similar products could not be used.